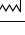
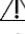
**'PRODUCT NAME: NASAL AND SINUS PACK**  
**'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

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**'PRODUCT NAME: NASAL AND SINUS PACK**
**'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

			medici o su prescrizione del medico NL - Waarschuwing: Volgens de Amerikaanse Federale wetgeving is verkoop van dit apparaat uitsluitend toegestaan door of op voorschrift van een arts. NO - Forsiktig: I henhold til føderale lover i USA skal dette utstyret bare selges av lege eller etter ordinasjon fra lege PT - Cuidado: A Legislação Federal dos EUA restringe a venda deste dispositivo a médicos ou mediante prescrição médica SV - Försiktighet: I USA får enligt federal lag denna produkt endast försäljas av läkare eller enligt läkares ordination				verpakking met inwendige steriele barrière NO - Usteril emballasje med sterile barriere på innsiden PT - Embalagem não estéril com barreira estéril no interior SV - Osterilt emballage med sterilbarriär inuti	
--	--	--	---	--	--	--	--	--

This Instructions for use relates to the following products:  
 GB - This Instructions for use relates to the following products:  
 DE - Diese Gebrauchsanweisung bezieht sich auf die folgenden Produkte:  
 DK - Denne brugsanvisning vedrører følgende produkter:  
 ES - Estas instrucciones de uso hacen referencia a los siguientes productos:  
 FI - Tämä käyttöohje liittyy seuraaviin tuotteisiin:  
 FR - Mode d'emploi des produits suivants :  
 IT - Le seguenti istruzioni per l'uso si riferiscono ai seguenti prodotti:  
 NL - Deze gebruiksaanwijzing heeft betrekking op de volgende producten:  
 NO - Denne bruksanvisningen gjelder for disse produktene:  
 PT - Estas instruções referem-se aos seguintes produtos:  
 SV - Denna bruksanvisning gäller följande produkter:

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**GB - INSTRUCTIONS FOR USE**
**Description**

The products in the Network range of PVA sponge products have been designed to meet the requirements of the surgeon where there is a requirement to staunch blood loss after invasive surgery or traumatic injury. The products are quick and easy to use and reduce the time required to create haemostasis where blood loss is critical. The products are available in a variety of designs, widths and lengths to best suit the surgeon's preference and needs. The range is further extended to accommodate for the requirements of children and adults of Asian origin.

**Intended Use**

PVA sponge ENT products are designed to staunch blood loss after invasive surgery or traumatic injury in ENT surgery. The PVA sponge reacts quickly to absorb blood and body fluids. PVA is biocompatible and in some instances can be used for post-operative tissue support and to carry medication to the operative site (where specified in the instructions for Use). Nasal Packs are intended for use after septal, Turbinate and Rhinoplastic surgery and provide gentle equal pressure within the nasal cavity when absorbing post-operative bleeding. Patient comfort can be enhanced by using an airway tube. The slimpack version nasal pack has been developed for use in conjunction with a nasal splint following septal surgery. Sinus Packs are used particularly as post-operative packing after Functional endoscopic sinus surgery, to prevent post-operative lateralisation of the middle turbinates. The packs can also be used to fill the osteomeatal complex to leave the nasal passage clear for easier breathing.

**CAUTIONS**

- This device is supplied STERILE and ready to use.
- This device is for SINGLE USE ONLY. Do NOT re-sterilise or re-use.
- Do not use if the packaging has been opened or damaged.
- This device is only to be used by a suitably trained or qualified person under the facility approved and recommended conditions.

**'PRODUCT NAME: NASAL AND SINUS PACK**

**'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

- The product comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination.
- CAUTION: US Federal law restricts this device to sale by or on the order of a physician.
- Do not leave packing in for more than 4 days, or serious side effects may occur.
- Always saturate packing before removal.
- String to be used as an indicator only.
- Securing the string to the patient's cheek will prevent the pack moving backwards, thus eliminating the possibility of the pack being imbedded further and causing an obstruction.
- The tubed products in this range contain silicone which may cause allergic reaction in patients with sensitivity to silicone. Risk assess the use of the devices in relation to the medical benefit of the procedure and take necessary precautions with patients with known sensitivity to silicone.

**Possible Adverse Effects**

Tampons are associated with Toxic Shock Syndrome (TSS). In case of symptoms such as sudden fever, vomiting, diarrhoea, dizziness, fainting or near fainting and for a rash appearing like a sunburn the treatment with NETCELL has to be interrupted immediately.

**Incident Reporting**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

**INSTRUCTIONS**

**Insertion**

All NETCELL packs may be trimmed before use (string must remain in packing).

Apply a water miscible antibiotic ointment (non-alcoholic) to all surfaces.

Hold the packing with forceps or fingers and gently insert to the desired packing area. (The entire packing should be within the nose).

Once in place, tilt the patient's head back and gradually irrigate with 5-10cc of saline or antibiotic solution to ensure full expansion.

The packing must remain moist while in use.

**Removal**

Always saturate all NETCELL packings with saline or sterile water before removal to ensure softness. If an airway is used, remove the tube first.

String is used as an indicator only.

Grasp the packing and gently remove.

Packing must be removed by a physician.

**Sterilisation**

This device is supplied sterile by Gamma Irradiation and are designated for SINGLE USE ONLY.

**HAZARDS ASSOCIATED WITH THE RE-USE OF SINGLE USE ONLY DEVICES**

1. Single use devices have not been validated for re-use.  
If you re-use a device you may be held Legally Liable for the safe performance.
2. Cross-contamination and infection risks to patients . Including transmission of:
  - CJD & Variant CJD.
  - Prion Diseases.
  - Bacterial Endotoxins.
  - Hepatitis B & Hepatitis C.
  - Risks posed by HIV and AIDS
3. Device failure through material fatigue or degradation caused by initial use and design.
4. Patient injury from device failure and/or chemical burns from residue of decontamination agents absorbed into the materials

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