




PVA Instrument Wipes Specification Sheet KITPACK (BULK) PRODUCT

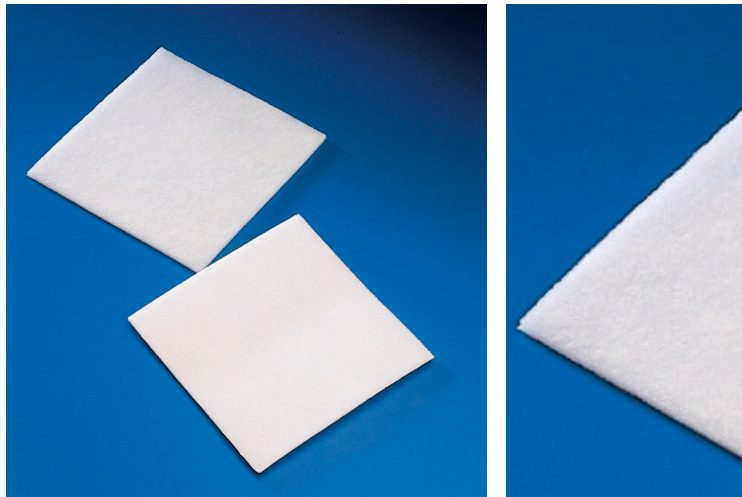

Product Overview

High quality and performance PVA Instrument wipe, not used in a clinical application on patients:

- Manufactured from ultra-smooth, biocompatible PVA sponge
- Soft, strong, and flexible
- Ultra-absorbent
- Supplied non sterile for direct inclusion in procedure packs
- Single use only, declared 5 year shelf life

The PVA Instrument Wipe is also available with the addition of a wick for the efficient management of fluids.



PVA Instrument Wipes Specification Sheet KITPACK (BULK) PRODUCT

PVA Instrument Wipe Options

Product Description	Pack Size	Unit of Sale	Dimensions	Product Code
Instrument Wipe	Glassine of 1	Bag of 250	3inch x 3inch x 2mm	1-900
Instrument Wipe	Bulk	Bag of 300	3inch x 3inch x 2mm	1-900B
Instrument Wipe	Bulk	Bag of 300	2.5inch x 2.5inch x 2mm	1-901B

Material Specification

Product Component	Specification
Instrument Wipe	100% Polyvinyl Alcohol Sponge (PVA)
Packaging	Paper glassine (for 1-900)

Intended Use

PVA instrument wipes are used to help clean medical devices after use.

Intended Purpose

A non-contact, single-use sheet of PVA intended to be used in a sterile condition by healthcare professionals to help clean medical devices prior to full decontamination. The instrument wipe is only to be handled with medical grade gloves and as such is non-contact with a cumulative transient use of less than 60 minutes. There are no known contraindications for this device. These devices are not intended to be used on a patient.

Sterilisation

Products are suitable for sterilisation by EO gas, gamma irradiation and e-beam. These devices are non-sterile and are suitable for Gamma Sterilisation at 25-35kGy. It is the responsibility of the procedure pack manufacturer to validate that their specific Ethylene Cycle does not adversely affect product performance. Network Medical can undertake product testing of customer sterilised product if required.

Instructions for Use

2 | Page



PVA Instrument Wipes Specification Sheet KITPACK (BULK) PRODUCT

The instructions for use are supplied in the form of a multi-lingual leaflet with instructions given. The international symbols used on the packaging are explained in each language. A leaflet is supplied with each product and is also available to view and download at <https://www.networkmedical.co.uk/ifu-product-group>

Conformity to the European Directives

EYETEC® Instrument Wipes are defined as non-invasive devices which do not touch the patient (Rule 1, Annex IX 93/42/EEC Medical Devices Directive) and are classified as Class I non-sterile.