Cellulose Spears	PVA Spears	PVA Points	PVA Wick	PVA Wick & Wipe	PVA Eye Drain	PVA LASIK Eye Drain	PVA Corneal Light Shield
PRODUCT DESCRIPTION							
Cellulose spears are triangular pieces of absorbent cellulose attached to a handle; intended to be used for the management of fluids on the exterior surface of the eye.	PVA spears are triangular pieces of absorbent Polyvinyl Alcohol (PVA) Foam attached to a handle; intended to be used for the absorption of fluids on the exterior surface of the eye.	PVA Points are triangular pieces of absorbent Polyvinyl Alcohol (PVA) Foam; intended to be used for the absorption of fluids on the exterior surface of the eye.	PVA Wicks are pieces of absorbent Polyvinyl Alcohol (PVA) Foam intended to be used for the management of fluids on the exterior surface of the eye.	PVA Wick and Wipes are made from Polyvinyl Alcohol (PVA) Foam in sheet form which can be used to manage fluids during ophthalmic procedures as well as clean medical instruments after use.	PVA Eye Drains are a rectangular strip of absorbent Polyvinyl Alcohol (PVA) Foam which is used for management of fluids during ophthalmic procedures. The fluid wicks into the drainage bag which has a self-adhesive strip to permit it to be secured to the drape.	The PVA LASIK Eye Drain is a piece of Polyvinyl Alcohol (PVA) Foam intended to be used during LASIK procedures for the management of fluids.	The PVA Corneal Light Shield is a disc of Polyvinyl Alcohol (PVA) Foam intended to shield the retina from intense operating light during ophthalmic procedures.
INTENDED USE/PURPOSE							
Single-use triangular piece of absorbent cellulose attached to a handle which is intended to be used for the management of fluids on the exterior surface of the eye.	Single-use triangular piece of absorbent PVA attached to a handle which is intended to be used for the absorption of fluids on the exterior surface of the eye.	Single-use triangular piece of absorbent PVA which is intended to be used for the absorption of fluids on the exterior surface of the eye.	Single-use piece of absorbent PVA which is intended to be used during ophthalmic procedures for the management of fluids.	This a dual purpose device, the smaller removable strip is a single-use piece of absorbent PVA which is intended to be used during ophthalmic procedures for the management of fluids and the larger piece can be used for cleaning medical instruments.	Single-use rectangular strip of absorbent PVA which is used for management of fluids during ophthalmic procedures. The fluid wicks into the drainage bag which has a self-adhesive strip to permit it to be secured to the drape.	Single-use piece of absorbent PVA which is intended to be used during LASIK procedures for the management of fluids.	Single-use PVA Corneal Light Shield which is intended to shield the retina from intense operating light during ophthalmic procedures.
INDICATIONS FOR USE							
Indicated for fluid management during ophthalmic procedures.							To shield the retina from intense operating light during ophthalmic surgery.
				WIPE ONLY: For the cleaning of medical instruments.			
CONTRAINDICATIONS							
They are contraindicated for patients undergoing LASIK surgery.	There are no known contraindications for these devices.						
SAFETY CLAIMS							
Provided sterile							
Wicks fluid away from the surgical field.				WICK ONLY: Wicks fluid away from the surgical field.	Wicks fluid away from the surgical field.		Helps protect the eye during ophthalmic procedures.
Shaped to enable improved access during procedures						Shaped to enable fluid management without impeding LASIK procedures.	Available in multiple sizes suitable for pediatric and adult patients.
CLINICAL BENEFIT							
Highly absorbent.				WICK ONLY: Highly absorbent.	Highly absorbent.		Available in multiple sizes suitable for pediatric and adult patients.
Wicks fluid away from the surgical field.				WICK ONLY: Wicks fluid away from the surgical field.	Wicks fluid away from the surgical field.		
Flexible handle aids placement of absorbent point.					WIPE ONLY: To clean medical instruments after surgical procedures prior to full decontamination.	Shaped to enable fluid management without impeding LASIK procedures. Wicks fluid away from the surgical field.	
PERFORMANCE CLAIMS							
Highly absorbent.				WICK ONLY: Highly absorbent.	Highly absorbent.		Helps protect the eye during ophthalmic procedures.
Wicks fluid away from the surgical field.				WICK ONLY: Wicks fluid away from the surgical field.	Wicks fluid away from the surgical field.		
Flexible handle aids placement of absorbent point.					WIPE ONLY: To clean medical instruments after surgical procedures prior to full decontamination.	Shaped to enable fluid management without impeding LASIK procedures.	

APPLICABLE TO ALL DEVICES LISTED IN THIS INSTRUCTION FOR USE	
INTENDED USERS	HUMAN BODY CONTACT
The device is intended to be used by appropriately trained healthcare professionals.	The devices directly contact the patient conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue.
USE ENVIRONMENT	PATIENT POPULATION
Hospital/clinical environment	Anyone of any age in the human population as determined by a suitably qualified healthcare professional undergoing ophthalmic procedures.
WARNINGS AND PRECAUTIONS	STERILISATION METHOD
The devices are supplied STERILE and ready to use. The devices are for individual patient and procedure SINGLE USE ONLY and should be disposed of after each use. Do NOT re-sterilise or re-use. Due to the design of the devices they are not suitable for re-sterilisation or re-use. There is a risk of cross-infection if reused. Do NOT use if the packaging has been opened or damaged. Loose particles may occur with Cellulose or PVA products, these should be rinsed from the treatment site with sterile water. Do NOT use excessive force as scratching of the ocular mucus membrane may occur. Any unused devices remaining in an opened pouch must be disposed of at the end of a procedure.	See outer labelling for method of sterilization
	STORAGE & HANDLING REQUIREMENTS
	These devices must be kept dry and stored out of direct sunlight There are no special handling instructions required to use these devices as indicated
	ADVERSE EVENTS
	No known adverse events from use of the device as indicated.
	DEVICE LIFETIME IN USE
	Single use transient devices with a continuous lifetime in use of less than 60 minutes.
OPERATIONAL USE	SHELF-LIFE
This is a standard device, for operational use please refer to indications and intended use.	The shelf life is 5 years.
DISPOSAL	INCIDENT REPORTING
After use the device should be disposed of in accordance with the healthcare settings disposal procedure. The product comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination.	Any incident that has occurred in relation to the device shall be reported to the manufacturer and appropriate medical device regulatory authority in which the user and/or patient is established.

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